

Levant

The Smooth, Quiet & Reliable Stair Lift



ThyssenKrupp Access



ThyssenKrupp

Levant Features and Benefits



Seat:

The ergonomically designed seat makes the ride smooth and comfortable. The seat width provides spacious seating. When folded up, the seat takes up significantly less space, so the stairs are free for other users.

The Levant has plenty of power with an ample lifting capacity of nearly 300 pounds. Your Levant dealer can provide you with more details if you have questions or concerns.

The Levant has a beautiful and clean look in any home because the colors of the seat, chair leg and lift drive are coordinated. It's designed to enhance your home's décor.

Padding:

We've created a firm but soft seat by using a thick, plush padding. The pad's neutral color fits any household and is fire retardant, liquid repellent and exceptionally easy to clean.

Seat Belt:

A simple "one-click" seat belt system provides even more safety and security and can be extended on both ends for further comfort.

Control Unit:

The Levant's control unit is easy to operate and is the same one used in more than 15,000 stair lifts worldwide. It's been proven reliable and is one of the Levant's many user-friendly features.

Swivel Handle:

This enables the chair to swivel to the safest position at the top of the stairs, which is particularly useful in tight spaces. With our swivel, you'll always move away from the stairs, increasing your personal safety. The handle is easy to operate and can be used on whichever side of the chair you prefer.





Footrest

The size of the footrest enables the user to place his feet comfortably next to each other. Along with the comfortable width, the color was also taken into consideration. The darker color on top of the footrest ensures a clean look for the chair lift — stains or smudges from shoes won't show easily on this darker gray color.

The footrest has several safety edges on both sides. When the footrest encounters an obstacle, the stair lift comes to an immediate stop, whether moving up or down.

Drive

The design of the drive shroud fits the chair and chair leg perfectly for a cohesive look. The embossed design is repeated on the drive, chair leg and even the Call/Sends, enhancing the Levant's sleek, unified appearance.

If you're on vacation or you know you won't be using your lift for any length of time, there's an on/off switch on your drive that will keep your chair battery from running down and extends your battery's life.

The Levant's battery has a long usage cycle and will provide you with mobility even during long power outages. For instance, our standard Levant lift will carry you 25 additional times even if nothing else in your house works!

And, as another safety precaution, special safety edges on the side of the drive are activated immediately when there is a trapping hazard, so neither you nor your stair lift will suffer any harm.

Call/Send

Two call/send stations are included with every unit. These are especially handy in a home in which more than one person uses the lift.

The call/sends have a slim design, with easy-to-use push buttons. The infrared control has a wide range, so the user does not have to point exactly at the drive. Just pointing toward the stair lift is more than enough.

Finally, the Levant's "hold to run" buttons ensure your safety during use. As soon as you release the button, the lift will stop.



Rail

The aluminum rail has a slim design with the rack securely covered, which gives it a modern look. In addition, the aluminum rail is anodized which keeps the surface clean. The rail supports are powder coated in a color which complements the aluminum.

The rail length can range from 1' to 24.5' (7.5 m) and the angle can be 28 to 45 degrees. We can fit the Levant to almost any straight run stairway.

Long charging contacts keep the lift batteries fresh and ensure that your chair will always stop at a charging point. An audible tone alerts you that your lift has arrived safely at its proper location.

An optional automatic folding rail is ideal for locations where the staircase ends in a doorway and the stair lift rail might pose a trip hazard. A minimum of 7' of travel is required for this feature.

Safety

In addition to the safety edges built into the drive and footrest that automatically stop the lift for unexpected objects on the staircase, the Levant has been rigorously tested and carries both the CE and ETL safety certification. You can use it with total confidence.

ThyssenKrupp Access Manufacturing, LLC

4001 East 138th Street
Grandview, MO 64030

Phone: 800.829.9760

Internet: www.tkaccess.com

Ascension Lift and Elevator Company
P.O. Box 62207
Colorado Springs, CO 80962
Colorado Springs - 719-264-9945
Denver - 720-375-4068

Email - info@ascensionlift.com

Website - www.ascensionlift.com